


Treatment guide for ANKLE JOINT PAIN with the Handy Pulse Laser

info@pulselaserrelief.com
1300 886 929

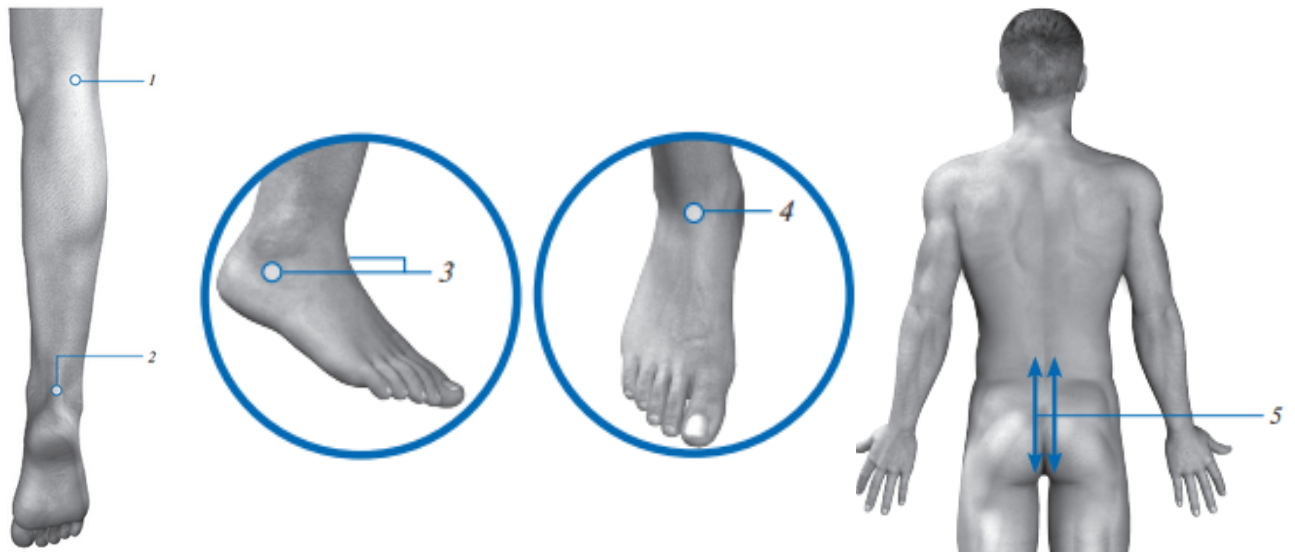

Session Procedure

No.	Exposure Area	Frequency	Exposure Time
1	Popliteal Pit	50 Hz	5 min
	Areas around talocrural joint:		
2	- rear projection of the joint (Achilles' tendon)	Variable mode 1000Hz to 2500Hz	2 min each per each zone
3	- side projections of the joint under the ankle on the right and on the left;	Variable mode 1000Hz to 2500Hz	2 min each per each zone
4	- front projection of the joint;	Variable mode 1000Hz to 2500Hz	2 min each per each zone
5	Along the lumbosacral spine (L4-Co), paravertebral, scanning	Variable mode 1000Hz to 2500Hz	2 min per side

Session frequency.....1 session per day or on alternative days

Course.....10-15 sessions

**Repeated courses should be based on indications, in 3-4 weeks.
Up to 3 laser therapy courses should be conducted during the chronic process.**