


Treatment guide for **SHOULDER PAIN BURSITIS** with the Handy Pulse Laser

info@pulselaserrelief.com
 1300 886 929

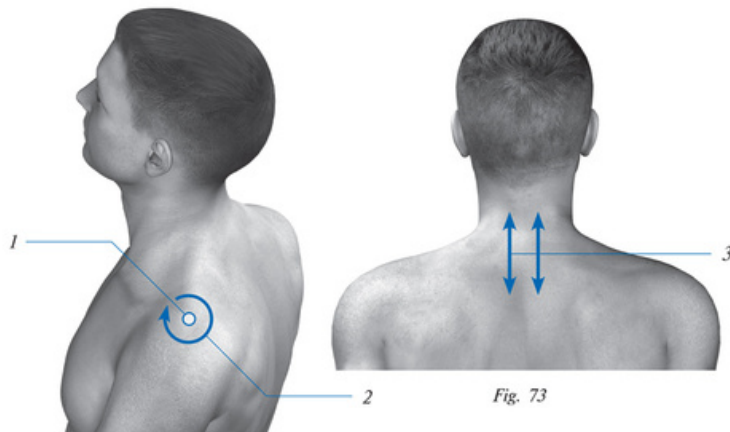

Session Procedure

No.	Exposure Area	Frequency	Exposure Time
1	Acromial process area	VARIABLE	5 min
2	5 cm around acromial process, scanning	VARIABLE	2 min
3	Along branches of the mandibular nerve (including the mental foramen, angle of mouth, tongue, gums, and teeth of lower jaw, skin of cheeks, lower jaw and lip and muscles of mastication), scanning.	VARIABLE	2 min
4	Along the cervical and thoracic departments of the spine (C5-Th1), paravertebral, scanning	VARIABLE	5 min on each side

Session Frequency: 1 session per day

Course: 10-15 sessions

Repeated courses should be based on indications, in 3-4 weeks.