


# Treatment guide for **SINUSITIS** with the Handy Pulse Laser

info@pulselaserrelief.com  
1300 886 929

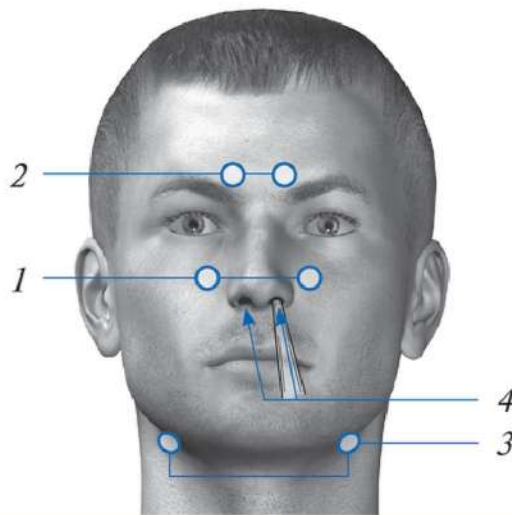

## Session Procedure

No.	Exposure Area	Frequency	Exposure Time
1	Maxillary sinus projections	5Hz	2 min on each side
2	Frontal sinus projections	5Hz	1 min on each side
3	Carotid arteries	50Hz	2 min on each side

- Laser therapy is strictly prohibited for purulent frontal sinusitis and rhinosinusitis (even though they are only suspected).**
- Laser therapy is conducted only after surgical treatment (puncture and drainage).**

Session frequency.....1 session per day

Course.....7-10 sessions

**Repeated courses should be based on indications, in 3-4 weeks.**