

Treatment guide for SPINAL AREA with the Handy Pulse Laser

info@pulselaserrelief.com
1300 886 929

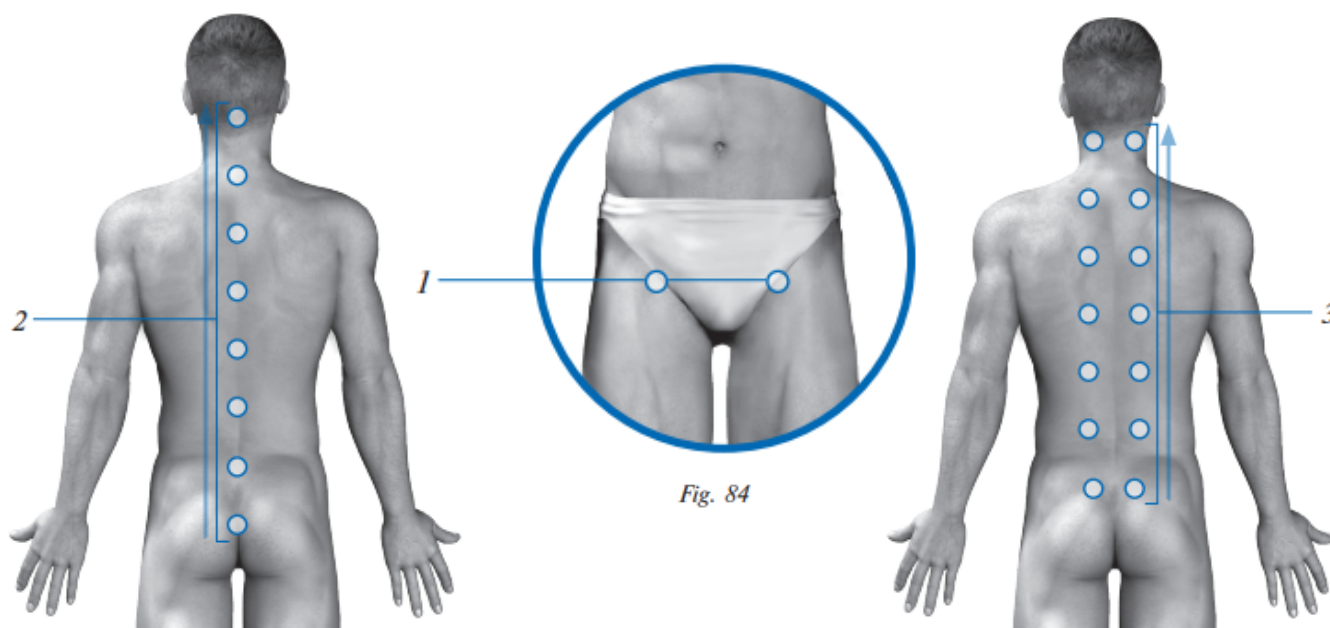


Fig. 84

Session Procedure

| No. | Exposure Area | Frequency | Exposure Time |
|-----|---|--|----------------|
| 1 | Femoral arteries | 50 Hz | 2 min per side |
| 2 | From coccyx to the nape, bottom to top along the acanthi, circa 8 zones (based on the height) at 10 cm interval | Variable Mode 1000Hz to 2500Hz | 1 min per area |
| 3 | Along the spine on the right and on the left, bottom to top along spinal extensor muscles at 10 cm interval | 50 Hz or VARIABLE (may be alternated) | 1 min |

Session frequency.....1 session per day or on alternative days

Course.....10-15 sessions

Repeated coursesin 3-4 weeks.