


Treatment guide for **ULNAR JOINT PAINS** with the Handy Pulse Laser

info@pulselaserrelief.com
1300 886 929

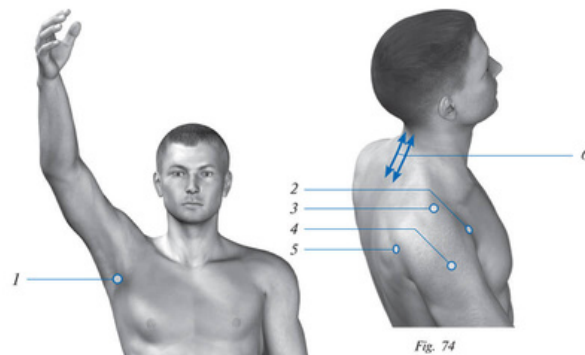

Session Procedure

No.	Exposure Area	Frequency	Exposure Time
1	Underarm pit	50Hz	5 min
2	Joint projection from the front	VARIABLE	2 min per each area
3	Joint projection from the top	VARIABLE	2 min per each area
4	Middle of deltoid	VARIABLE	2 min per each area
5	Joint projection from the rear	VARIABLE	2 min on each side
6	Along the neck and chest section of the spine (C5-Th1), paravertebral, scanning	VARIABLE	2 min on each side

Session Frequency: 1 session per day or on alternative days

Course: 10-15 sessions

Repeated courses should be based on indications, in 3-4 weeks.

4-6 laser therapy courses are permitted per year.